



Zio XT – a new remote service for ambulatory ECG in response to the COVID-19 crisis

Glyn Barnes



Zio XT homecare delivery service

- 1 – 14 day continuous ECG monitoring – an adhesive ‘patch’
- Delivered to the patient’s home
- Returned to iRhythm office
 - Reply-paid box provided
- No hospital visit necessary
- No risk from cross infection
 - A new device is sent to each patient
- No need to decontaminate



- Patient support materials provided:
 - ‘How to apply’ booklet
 - Online application video
 - 24/7 phone support

Zio XT homecare delivery service

- Uses proprietary AI algorithm for initial analysis
- Final detailed report composed by iRhythm's own accredited ECG technicians in the UK
- Report completed within 1 – 4 days
- Posted on iRhythm's web-based platform, Zio Reports, as a pdf
- View, print and save



Zio XT

- High throughput of patients
- Event button for patient to trigger
- No battery charging
- Showerproof
- Over 30 published papers
- Latex free
- 95% of product is recycled
 - Internal board and batteries



The Zio homecare pathway

Patient has consultation with GP or cardiologist



Doctor prescribes Zio for 1 – 14 days



Doctor/clinic emails patient request form to iRhythm



iRhythm returns form with Zio serial number contained



Doctor/clinic logs patient with serial number into Zio Reports



iRhythm sends device to patient's home address



Patient applies Zio with support from iRhythm



Patient removes Zio after prescribed time and sends to iRhythm



iRhythm analyses and posts report online in Zio Reports (email sent to doctor)



Doctor reviews result and begins care plan



Pricing offer during the COVID-19 crisis

- In response to the COVID crisis, iRhythm is now offering a reduced price of **£199** incl. (normal price £295). There are no postage costs and VAT is not applicable as Zio is regarded as a service
- For the purposes of this AHSN project, iRhythm has now waived any volume related restrictions to the above price